

CHAPTER 3

TEST DAY SET UP

Collector tapes appropriate signs on bathroom doors and inside stall doors being used

Collector secures the toilet to be used for donor's use and puts bluing in the water of all restroom toilets

Collector secures all water supplies, possible contaminants, openings, and any toilets or stalls not being used during the testing process with evidence tape

**Tape one of these signs to
the door of each restroom
being used for testing**



**RESTROOM CLOSED
FOR DFCA TESTING**



RESTROOM CLOSED

FOR DFCA TESTING

Tape one of these signs to the inside of the door of each stall being used for testing

Don't be nervous, this is easy...

1. Fill your cup ½ way with urine. (pee in the cup.)
2. **Set Your Cup Down.**
3. Go get the collector. (Leave cup in restroom.)

 **DO NOT**
Flush the toilet.

 **DO NOT**
Wash Your Hands.

Drug Free Clubs
OF AMERICA 

© Copyright Drug Free Clubs of America, Inc. 2015, all rights reserved

Don't be nervous, this is easy...

1. Fill your cup ½ way with urine. (pee in the cup.)

2. **Set Your Cup Down.**

3. Go get the collector. (Leave cup in restroom.)



DO NOT

Flush the toilet.



DO NOT

Wash Your Hands.

Drug Free Clubs
OF AMERICA 